

# SunOne™ SYSTEM

One time. One choice. One solution.

Disposable Laryngoscopes by SunMed

## GreenLine®/D™ LED Disposable Laryngoscope Handle

Fiber Optic Green System Compatible

Used for endotracheal intubation

### SUGGESTED PROCEDURE

1. Remove handle from pouch
2. Slide heel of blade on the handle
3. Raise tip of blade to a right angle to illuminate
4. Ensure blade is fully engaged on handle before use
5. Do not use any disinfectant on handle
6. Do not process or disassemble

### HOJAS DESECHABLES DE LARINGOSCÓPIO

1. Abra paquete y remueva mango
2. Deslice base de la hoja en el mango
3. Levante la hoja formando 90° hasta que ilumine
4. Verifique que la hoja este bien sujeta al mango antes de usar
5. No use desinfectantes en el mango
6. No procese o desarme

Not Made  
with Natural  
Rubber Latex



For Professional  
Use Only

STERILE EO

Sterilized Using  
Ethylene Oxide



Do Not  
Resterilize



Single  
Use



Do Not Use  
if Package  
is Damaged



Consult IFU



Manufactured for SunMed  
2710 Northridge Dr. NW Suite A  
Grand Rapids, MI 49544 USA  
[www.Sun-Med.com](http://www.Sun-Med.com)

SunMed is a registered trademark in the USA  
Made in China

DisposableHandle.IFU Rev.1